

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 65376 Part Number: HIV9089

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ORTHO VITROS Eci ANTI-HIV 1/2 TEST DATE: UNK	CHIRON HIV-1 RNA bDNA TEST DATE: UNK	ABBOTT HIV-1 p24 Ag TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK	ABBOTT HIV-1 p24 Ag NT% TEST DATE: UNK
9089-01	11/09/97	0.14	<50	0.28	0.06	0.06	0.00	NEGATIVE
9089-02	11/16/97	0.12	<50	0.35	0.06	0.06	0.02	NEGATIVE
9089-03	11/18/97	0.13	316	0.32	0.09	0.09	0.02	NEGATIVE
9089-04	11/25/97	1.02	146954	2.88	24.93	24.93	0.00	100.2
9089-05	11/29/97	29.1	59647	0.77	6.45	6.45	0.01	113.5
9089-06	12/03/97	37.8	42837	1.25	5.72	5.72	0.45	124

PANEL MEMBER	GENETICS SYSTEMS HIV ½ EIA TEST DATE: UNK	COULTER HIV-1 p24 Ag TEST DATE: UNK	INNOTEST ANTI-HIV 1/2½ TEST DATE: UNK	ABBOTT HIV AB ½ EIA rDNA IMx TEST DATE: UNK	ABBOTT AxSYM HIV AB ½ EIA TEST DATE: UNK	ABBOTT HIV AB ½ EIA rDNA TEST DATE: UNK	ABBOTT MUREX HIV AB ½ + O 3 rd Generation TEST DATE: UNK	ABBOTT HIV AB ½ EIA rDNA 3 rd Generation TEST DATE: UNK
9089-01	0.173	0.233	0.067	0.28	0.32	0.234	0.428	0.21
9089-02	0.211	0.233	0.043	0.30	0.35	0.211	0.386	0.19
9089-03	0.198	0.233	0.050	0.31	0.33	0.234	0.419	0.22
9089-04	0.201	8.023	0.063	0.35	0.33	0.313	0.590	0.17
9089-05	0.188	2.910	0.050	1.25	0.73	1.211	9.673	0.44
9089-06	0.355	2.910	2.286	4.69	2.35	4.422	10.354	2.33

PANEL MEMBER	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: UNK	ABBOTT PRISM HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK	ABBOTT MUREX MICROPLATE HIV AG/AB COMBO 4 th Generation TEST DATE: UNK	ABBOTT HIV AG/AB COMBO ARCHITECT 4 th Generation TEST DATE: UNK	ABBOTT PRISM ANTI-HIV ½ TEST DATE: UNK	BIOTEST ANTI-HIV ½ TEST DATE: UNK
9089-01	0.10	0.09	0.281	0.11	0.34	0.06
9089-02	0.03	0.10	0.291	0.12	0.38	0.08
9089-03	0.02	0.17	0.305	0.11	0.37	0.09
9089-04	0.67	4.31	3.897	4.33	0.33	0.13
9089-05	9.99	2.09	4.103	1.01	0.43	1.14
9089-06	10.94	3.35	10.749	2.24	1.63	4.89

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9089 Rev06
Effective Date: 06/15/2021