



HIV Seroconversion Panel Donor No. 1043487
Part Number: HIV9096

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ORTHO VITROS ECI ANTI-HIV 1/2	VENDOR HIV ½	ABBOTT HIV ½	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 2/6/2010	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9096-01	11/26/06	0.13	NEGATIVE	0.17	0.15	0.15	0.01
9096-02	11/29/06	0.14	NEGATIVE	0.25	2.41	2.41	0.05
9096-03	12/03/06	0.15	NEGATIVE	0.17	112.81	112.81	0.12
9096-04	12/06/06	6.52	NEGATIVE	0.33	124.12	124.12	0.10
9096-05	12/14/06	52.6	POSITIVE	10.50	83.69	11.51	83.69
9096-06	12/17/06	40.5	POSITIVE	5.42	133.39	NOT REACTIVE-HIGH-AB	133.39

PANEL MEMBER	GENETICS SYSTEMS HIV ½ Plus O	GENETICS SYSTEMS HIV rLav	CAMBRIDGE BIOTECH HIV WESTERN BLOT	TRINITY UNIGOLD	BIO-RAD HIV MULTISPOT	CHEMBIO HIV STAT PAK	CHEMBIO HIV SURE CHECK	ROCHE HIV MONITOR
	TEST DATE: 4/12/2010	TEST DATE: 4/12/2010	TEST DATE: 4/12/2010	TEST DATE: 4/12/2010	TEST DATE: 4/12/2010	TEST DATE: 4/12/2010	TEST DATE: 4/12/2010	TEST DATE: 4/19/2010
9096-01	0.11	0.15	0	0	0	0	0	934
9096-02	0.15	0.08	0	0	0	0	0	40500
9096-03	0.15	0.04	0	0	0	0	0	1370000
9096-04	0.15	0.08	0	0	0	0	0	1660000
9096-05	11.44	4.5	+/-	+1	+2	+1	+1	334000
9096-06	11.44	5.04	+2	+1	+2	+2	+2	187000

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com