

ZeptoMetrix®

Syphilis Mixed Titer Panel

Part Number: KZMC002

Lot# 0709-272-00053

Donor Demographics: The syphilis positive donor plasmas used for this panel were obtained under informed consent and collected in US FDA licensed plasma centers. Donors were untreated and asymptomatic at the time of collection.

Member Number	Date of Draw	Blood Number	Vendor HIV p24 Ag Test Date: 5/1999	Vendor ANTI HIV ½ Test Date: 5/1999	Vendor HBsAg Test Date: 5/1999	Vendor ANTI HCV Test Date: 5/1999	Vendor RPR Test Date: 5/1999
K-ZMC002-01	09/18/1996	N-10643-093	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-02	07/11/1996	N-86246-062	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-03	07/07/1996	N-75357-096	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-04	07/08/1996	N-40538-063	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-05	09/23/1996	N-56902-139	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-06	09/24/1996	N-55123-020	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-07	07/23/1996	G-54655-081	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-08	08/19/1996	G-85547-014	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-09	10/13/1996	N-44833-014	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-10	07/26/1996	N-13962-041	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-11	06/28/1996	G-67031-036	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-12	09/11/1996	N-45745-018	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-13	10/09/1996	N-12969-079	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-14	07/26/1996	N-92228-039	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE
K-ZMC002-15	08/07/1996	N-14210-108	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE

Member Number	Becton Dickinson RPR Test Date: 5/1999	Trinity Captia IgM - S/Co Test Date: 9/2007	Trinity Captia IgG - S/Co Test Date: 9/2007	Fisher Sure Vue RPR Test Date: 9/2007	Inverness/Wampole FTA-ABS SYPHILLIS Test Date: 9/2007
K-ZMC002-01	Not Tested	0.29	2.49	POSITIVE	POSITIVE
K-ZMC002-02	1:8	0.43	1.61	POSITIVE	POSITIVE
K-ZMC002-03	1:2	0.50	1.35	POSITIVE	POSITIVE
K-ZMC002-04	1:8	0.36	3.16	POSITIVE	POSITIVE
K-ZMC002-05	Not Tested	0.36	2.95	POSITIVE	POSITIVE
K-ZMC002-06	1:4	0.73	2.34	POSITIVE	POSITIVE
K-ZMC002-07	1:8	1.02	3.28	POSITIVE	POSITIVE
K-ZMC002-08	1:2	0.36	1.6	POSITIVE	INDETERM
K-ZMC002-09	1:2	0.15	1.93	POSITIVE	POSITIVE
K-ZMC002-10	Not Tested	1.24	3.7	POSITIVE	POSITIVE
K-ZMC002-11	1:4	0.95	1.34	POSITIVE	POSITIVE
K-ZMC002-12	Not Tested	1.17	2.27	POSITIVE	POSITIVE
K-ZMC002-13	1:4	0.36	2.44	POSITIVE	POSITIVE
K-ZMC002-14	Not Tested	0.95	4.54	POSITIVE	POSITIVE
K-ZMC002-15	Not Tested	0.29	2.61	POSITIVE	POSITIVE

This data is intended to be representative of typical test procedures and should be used for informational purposes only. They are not intended to represent performance specifications.

Notes on Data/Panel:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. These specimens are not a substitute for the positive and negative control reagents provided with licensed test kits.
6. The Syphilis Mixed Titer Panel is a 15 member panel consisting of vials each containing 1mL of plasma.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIKZMC002 Rev04
 Effective Date: 11/23/2021

ZeptoMetrix®

Syphilis Mixed Titer Panel

Part Number: KZMC002

Lot# 0709-272-00053

Donor Demographics: The syphilis positive donor plasmas used for this panel were obtained under informed consent and collected in US FDA licensed plasma centers. Donors were untreated and asymptomatic at the time of collection.

Member Number	Fisher Sure Vue RPR Test Date: 1/2019	Phoenix BioTech Trep Sure - S/Co Test Date: 1/2019	Fugirebio Diagnostic Serodia TPPA Test Date: 1/2019
K-ZMC002-01	POSITIVE	3.89	POSITIVE
K-ZMC002-02	POSITIVE	3.90	POSITIVE
K-ZMC002-03	POSITIVE	3.83	POSITIVE
K-ZMC002-04	POSITIVE	3.85	POSITIVE
K-ZMC002-05	NEGATIVE	3.82	POSITIVE
K-ZMC002-06	POSITIVE	3.79	POSITIVE
K-ZMC002-07	POSITIVE	3.82	POSITIVE
K-ZMC002-08	NEGATIVE	3.89	POSITIVE
K-ZMC002-09	POSITIVE	3.76	POSITIVE
K-ZMC002-10	POSITIVE	3.76	POSITIVE
K-ZMC002-11	POSITIVE	3.82	POSITIVE
K-ZMC002-12	POSITIVE	3.83	POSITIVE
K-ZMC002-13	POSITIVE	3.83	POSITIVE
K-ZMC002-14	POSITIVE	3.86	POSITIVE
K-ZMC002-15	POSITIVE	3.40	POSITIVE

This data is intended to be representative of typical test procedures and should be used for informational purposes only. They are not intended to represent performance specifications.

Notes on Data/Panel:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. These specimens are not a substitute for the positive and negative control reagents provided with licensed test kits.
6. The Syphilis Mixed Titer Panel is a 15 member panel consisting of vials each containing 1mL of plasma.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIKZMC002 Rev04
Effective Date: 11/23/2021