

PRODUCT DESCRIPTION:

NATtrol™ SARS-CoV-2 Omicron External Run Control* is formulated with purified, intact viral particles. The virus particles have been chemically modified to render them non-infectious and refrigerator stable**. Each control pack contains 6 x 0.5 mL of NATtrol™ SARS-CoV-2. This control is provided in a proprietary matrix.

*Pat.: <http://www.zeptometrix.com/patent-information/>

INTENDED USE:

- NATtrol™ SARS-CoV-2 Omicron External Run Control is designed to evaluate the performance of nucleic acid tests for determination of the presence of SARS-CoV-2 Omicron Variant RNA. NATCOV2(OMC)-ERC enables laboratories to monitor test variation, lot-to-lot test kit performance, operator variation, and can provide assistance in identifying random or systemic error.

WARNINGS AND PRECAUTIONS:

- NATtrol™ inactivation was carried out on SARS-Cov-2 stock used to formulate the control. The inactivation was verified in a standard microbiological growth protocol.
- This product contains inactivated microorganisms and materials of human and animal origin. Safe practices suggest that the controls be considered potentially infectious and to use Universal Precautions when handling.
- Refer to CDC guidelines and local regulations for handling and disposal.
- The matrix used in the manufacture of this product is treated with 0.09% sodium azide. It was manufactured from Human Serum Albumin that has been tested and found to be non-reactive at the donor level for HIV-1/HIV-2 Antibody, HBsAg and HCV Antibody by FDA licensed donor screening test methods. All materials are also tested for HIV-1 and HCV by FDA approved Nucleic Acid Test (NAT) methods.
- Heat inactivated Fetal Bovine Serum used in the manufacture of this product meets applicable USDA requirements for abattoir sourced animals, traceability and country of origin. The materials were collected at USDA licensed establishments or legally imported from countries recognized by the USDA as negligible or controlled for risk for Bovine Spongiform Encephalopathy (BSE) and other exotic disease agents. Donor animals were inspected ante and post mortem at the abattoir as required by the USDA.
- Do not use past the expiration date on the label.
- To avoid cross-contamination, use separate pipette tips for all materials.

RECOMMENDED STORAGE:

- NATtrol™ SARS-CoV-2 Omicron External Run Control should be stored at 2-8°C.

INSTRUCTIONS FOR USE:

- Mix tube vigorously for at least 5 secs.
- Process according to manufacturer's instructions for sample to result assays.
- Extract nucleic acid prior to use in downstream assays that are not sample to result.

LIMITATION:

- FOR RESEARCH USE ONLY. NOT FOR USE IN DIAGNOSTIC PROCEDURES**
- Quality control materials should be used in accordance with local, state, federal, and accreditation requirements.
- This product is not intended to replace the manufacturer's controls provided with the assay.

EXPECTED RESULTS:

- Each laboratory must evaluate the product and establish their own acceptance criteria.
- Virus information and target level listed in table below.








TABLE 1:

Catalog Number	Virus/Strain	Target Concentration
NATCOV2(OMC)-ERC	SARS-CoV-2 Omicron Variant; Lineage: B.1.1.529 (Isolate: USA/MD-HP20874/2021) ¹	50,000 copies/mL ²

¹ This reagent was obtained through BEI Resources, NIAID, NIH: Severe Acute Respiratory Syndrome-Related Coronavirus 2, Isolate hCoV-19/USA/MD-HP20874/2021 (Lineage B.1.1.529; Omicron Variant), NR-56461, contributed by Dr. Andrew Pekosz.

² Quantitation based on in-house RT-PCR assay targeting N gene region. Quantitation standard assigned copies/mL by dPCR.

PINATCOV2(OMC)-ERC
Revision: 00
Effective Date: 03/03/2022

	Catalog Number		Temperature Limitation
	Batch Code		Expiration Date
	For Research Use Only		Biological Risk
	Manufacturer		

www.ZeptoMetrix.com

ZeptoMetrix LLC • 878 Main Street, Buffalo, NY 14202 USA • Tel (800) 274-5487

This product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.