

NATtrol™ Epstein-Barr Virus External Run Controls Strain: B95-8

Catalog Number: NATEBV-ERCL
Catalog Number: NATEBV-ERCM

PRODUCT DESCRIPTION:

NATtrol™ EBV External Run Controls (NATEBV-ERCL and NATEBV-ERCM)* are formulated with purified, intact viral particles that have been chemically modified to render them non-infectious and refrigerator stable. Each control pack contains 6 X 1.0mL of NATtrol™ Epstein-Barr Virus. These controls are provided in a proprietary matrix.

*Pat.: <http://www.zeptometrix.com/patent-information/>

INTENDED USE:

- NATtrol™ EBV external run controls are designed to evaluate the performance of nucleic acid tests for determination of the presence of EBV nucleic acid. NATEBV-ERCL and NATEBV-ERCM enable laboratories to monitor test variation, lot-to-lot test kit performance, operator variation, and can provide assistance in identifying random or systemic error.

WARNINGS AND PRECAUTIONS:

- NATtrol™ inactivation was carried out on EBV stocks used to formulate the controls. The inactivation was verified by the absence of viral growth of a surrogate virus in validated tissue culture based infectivity assays.
- This product contains inactivated microorganisms and materials of human and animal origin. Safe practices suggest that the controls be considered potentially infectious and to use Universal Precautions when handling.
- Refer to CDC guidelines and local regulations for handling and disposal.
- The matrix used in the manufacture of this product is treated with 0.09% sodium azide. It was manufactured from Human Serum Albumin that has been tested and found to be non-reactive at the donor level for HIV-1/HIV-2 Antibody, HBsAg and HCV Antibody by FDA licensed donor screening test methods. All materials are also tested for HIV-1 and HCV by FDA approved Nucleic Acid Test (NAT) methods.
- Heat inactivated Fetal Bovine Serum used in the manufacture of this product meets applicable USDA requirements for abattoir sourced animals, traceability and country of origin. The materials were collected at USDA licensed establishments or legally imported from countries recognized by the USDA as negligible or controlled for risk for Bovine Spongiform Encephalopathy (BSE) and other exotic disease agents. Donor animals were inspected ante and post mortem at the abattoir as required by the USDA.
- Do not use past the expiration date on the label.
- To avoid cross-contamination, use separate pipette tips for all materials.

RECOMMENDED STORAGE:

- NATtrol™ EBV External Run Controls should be stored at 2-8°C.

INSTRUCTIONS FOR USE:

- Mix tube vigorously for at least 5 secs.
- Process according to manufacturer's instructions for sample to result assays.
- Extract nucleic acid prior to use in downstream assays that are not sample to result.

LIMITATION:

- FOR RESEARCH USE ONLY. NOT FOR USE IN DIAGNOSTIC PROCEDURES**
- Quality control materials should be used in accordance with local, state, federal, and accreditation requirements.
- This product is not intended to replace the manufacturer's controls provided with the assay.

EXPECTED RESULTS:

- Each laboratory must evaluate the product and establish their own acceptance criteria.
- The table shown below is for informational purposes only.

TABLE 1:

Catalog Number	Target Concentration (copies/mL) ^{1,2}
NATEBV-ERCL	5,000
NATEBV-ERCM	50,000

¹Quantitation based on in-house real time PCR assay targeting the LMP-2B gene region.

²Conversion factor: 1 copy = 1.62 IU. Based on internal testing of the 1st WHO International Standard for Epstein-Barr Virus for Nucleic acid Amplification Techniques (NIBSC code: 09/260)

ZMC-PI-0020
Revision: 10
Effective Date: 02/09/2022

	Catalog Number		Temperature Limitation
	Batch Code		Expiration Date
	For Research Use Only		Biological Risk
	Manufacturer		

www.zeptometrix.com

ZeptoMetrix LLC • 878 Main Street, Buffalo, NY 14202 USA • Tel (800) 274-5487

This product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

PCA# 20-223 & 21-112
Page 1 of 1