

PRODUCT DESCRIPTION:

NATtrol™ MRSA/SAP Panel (NATMRSA/SAP-C) is formulated with purified, intact bacterial particles that have been chemically modified to render them non-infectious and refrigerator stable*. NATMRSA/SAP-C contains 5 x 0.5 mL vials of *S. aureus* or *S. epidermidis* NATtrol™ as listed in Table 1. These panels are supplied in a purified protein matrix that mimics the composition of a true clinical specimen.

*NATtrol™ Patents Pending

INTENDED USE:

- NATtrol™ MRSA/SAP Panel is designed to evaluate the performance of nucleic acid tests for determination of the presence of *S. aureus* DNA. NATMRSA/SAP-C can also be used for verification of clinical assays, development of diagnostic tests and training of laboratory personnel.
- NATMRSA/SAP-C contains intact organisms and should be run in a manner identical to that used for clinical specimens.

ETIOLOGIC STATUS/BIOHAZARD TESTING:

- NATtrol™ inactivation was carried out on the bacterial stock used to formulate panel members. The inactivation was verified by the absence of bacterial growth in a validated growth protocol.
- Purified protein matrix used in the manufacture of this product is treated with 0.09% sodium azide. It was manufactured from materials that have been tested and found non-reactive at the donor level for HIV-1/HIV-2 Antibody, HBsAg and HCV Antibody by FDA licensed donor screening test methods. All materials are also tested for HIV-1 and HCV by FDA approved Nucleic Acid Test (NAT) methods.

PRECAUTIONS:

- Although NATMRSA/SAP-C contains inactivated organisms, it should be handled as if potentially infectious.
- Use Universal Precautions when handling this product.
- To avoid cross-contamination, use separate pipette tips for all reagents.

RECOMMENDED STORAGE:

- NATtrol™ MRSA/SAP-C should be stored at 2-8°C.

INSTRUCTIONS FOR USE:

- Extract Nucleic Acids prior to use in downstream assays. Follow manufacturer instructions for integrated sample to result assays.

Table 1: PANEL MEMBERS:

Panel Member	Strain
<i>S. aureus</i>	MRSA Community Strain
<i>S. aureus</i>	MRSA Hospital Strain
<i>S. aureus</i>	MSSA – empty cassette
<i>S. aureus</i>	MSSA
<i>S. epidermidis</i>	MSSE HER 1292

DO NOT USE IN HUMANS:

These products are intended for research, product development, quality assurance or manufacturing use. These products are NOT intended for use in the manufacture or processing of injectable products subject to licensure under section 351 of the Public Health Service Act or for any other product intended for administration to humans.

This product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.

PINATMRSA/SAP-C
Rev. No./Replaces: 2 / 09/2013
Research Use Only

ZeptoMetrix Corporation
878 Main Street
Buffalo, NY 14202
800-274-5487
www.zeptometrix.com

PCA No.: 15-204